CYAN

Washing Solution

Solution de Lavage

Code: CY001-WS 1 x 100 ml

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Intended use

The CYAN Washing Solution is a neutral surfactant that can be used for everyday cleaning of CYANStart, CYANSmart, CYANPro and CYANVision biochemistry analysers.

Preparation

Make a 5% (v/v) dilution of CYAN Washing Solution. To prepare 1 L: add 50 mL of CYAN Washing Solution to 950 mL distilled water.

This solution can be stored for several months at room temperature. Discard the solution if it is contaminated by bacteria or mould.

Washing Procedure

At the end of each working day, it is recommended to rinse the instrument with a neutral detergent solution like CYAN Washing Solution.

CYANStart/Smart

Aspirate 5 mL of diluted CYAN Washing Solution by placing a tube under the aspiration tube and pressing WASH. Afterwards, aspirate 5 mL of distilled water, to rinse the flowcell. As last, aspirate air until you see the air coming out of the waste tube. This is done to avoid microbial contamination during stand-still.

CYANVision

Place a container with at least 15 mL diluted CYAN Washing Solution under the aspiration inlet. Launch the Cleaning procedure from the Maintenance - Overview menu, press aspirate to start. Remove the CYAN Washing Solution and press aspirate to repeat the operation with air. Afterwards, aspirate 15 mL distilled water to wash the hydraulic system. Aspirate air again to dry the system and avoid microbial contamination during stand-still.

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